




Dipyrone

Metamizole

Analgesics

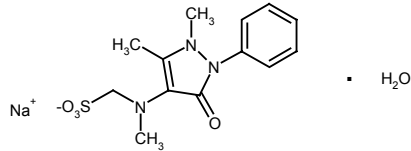

$C_{13}H_{16}N_3NaO_4S \cdot H_2O$
 MW 351.4
 sodium salt

A Commitment to Diagnostics

Order Code BAG2-C59

Generic Names Acefalgin; Alginodia; Algocalmin; Analgin; Andolor; Ankaljin; Acefalgin; Analgin; Andolor; Ankaljin; Antalgin Marsan; Baralgin M; Bebealjin; Bebigut; Beserol NF; Bonpyrin; Conmel; Devalgin; Di-Bal-Rone; Dinador; Dipiron; Dipyrivo; Divarine; Dolazon; Dolemicin; D-Pron; Dumatgin; Dya-Tron; Espyre; Everalgin; Farmolisina; Febralgine; Feverall; Fevonil; Keypyrone; Lagalgin; Lunalgine; Magnemidon; Metapyrin; Metilon; Minalgin; Narone; Nartate; Neomelubrin; Neuro-Fortamin; Nevalgina; Nobelgin; Nolotil; Norgesic; Novacid; Novalcina; Novaldin; Novalgetol; Novalgine; Novalgine; Novaminsulfon; Novemina; Novil; Optalgin; Paralgin; Pyralgin; Pylil; Pylilgin; Pyrojec; Repriman; Sebon; Spasmin; Sulfonovin; Tapal; Tega-Pyrone; Temp; Toloxin; Unagen; Vetalgin

Instruction for use Use a fresh allergen vial each time a new stimulation is performed
 Dissolve the allergen prior to use as follows:
 Add 250 µl of Stimulation Buffer to vial, vortex briefly
 Use allergen solution according to the protocol

Recommendation Additional to the normal concentration of the allergen given, we recommend to stimulate the cells with 1:5 diluted allergen solution
 (Mix 40 µl of allergen solution with 160 µl of stimulation buffer)

n Stimulation Up to 4 stimulations could be run with 1 vial

Storage Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial			
12.5 µg			
Conc. in vial			
50 µg/ml			
Conc. in stimulation	10 µg/ml	25 µg/ml	11.5 µg/ml
Cut-off	≥ 50 pg/ml	≥5% CD63 SI* (≥2)	≥5% CD63 SI* (≥2)

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays
 *SI (Stimulation Index) is defined as the gross stimulation divided by the negative control value



BÜHLMANN Laboratories AG
 Baselstrasse 55
 CH-4124 Schönenbuch/Basel
 Switzerland

Phone +41 61 487 12 12
 Fax orders +41 61 487 12 99
 info@buhlmannlabs.ch
 www.buhlmannlabs.ch



ADS014-02E_C59

CAST® is a registered trademark of BÜHLMANN.