



Neuromuscular Blockers Mix

Anaesthetics

A Commitment to Diagnostics

Name	Order Code	Common Name				
		EN	FR	DE	IT	ES
Neuromuscular blockers Mix	BAG2-CNBX	Mélange de Myorelaxants	Muskelrelaxantien		Mix de miorelassanti	Mix de relajantes musculares

Components	Code	MW	Content per vial	Conc. after rec.
Suxamethonium	CSUX	361	795 µg	3180 µg
Rocuronium	CROC	610	43 µg	171 µg
Vecuronium	CVEC	638	96 µg	383 µg
Pancuronium	CPAN	733	95 µg	381 µg
Atracurium	CATR	1243	12 µg	50 µg
Mivacurium	CMIV	1029	10 µg	41 µg
Cisatracurium	CCAT	929	20 µg	82 µg
Total			1072 µg	4286 µg

Instruction for use Use a fresh allergen vial each time a new stimulation is performed

Dissolve the allergen prior to use as follows:

Add 250 µl of Stimulation Buffer to vial, vortex briefly

Use allergen solution according to the protocol

N Stimulation 1 allergen vial is sufficient for up to 4 stimulations

Storage Store unopened allergens at 2-8°C until exp. date

Recommendation Additional to the normal concentration of the allergen given, we recommend to stimulate the cells with 1:5 diluted allergen solution (Mix 40 µl of allergen solution with 160 µl of stimulation buffer)

Content per vial 1.1 mg

Conc. after Rec. 4.3 mg/ml

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Conc. in stimulation	0.86 mg/ml	2.1 mg/ml	0.97 mg/ml
Cut-off	200 pg/ml undiluted 75 pg/ml 1:5 diluted	n.d.	≥5% CD 63

CAST®-Allergens are designed for in vitro leucocyte stimulation in the CAST®-Assays



BÜHLMANN Laboratories AG
Baselstrasse 55
CH-4124 Schönenbuch/Basel
Switzerland

Phone +41 61 487 12 12
Fax orders +41 61 487 12 99
info@buhlmannlabs.ch
www.buhlmannlabs.ch



ADS014-04E_CNBX

CAST® is a registered trademark of BÜHLMANN