




Glutamate

Food Additives

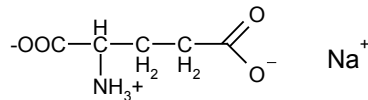

$\text{C}_5\text{H}_8\text{NNaO}_4$
MW 187.1

A Commitment to Diagnostics

Order Code BAG2-CE621

Use Flavor enhancer; swine food additive; with sugar, to improve palatability of bitter drugs

Generic Names E621; Chinese seasoning; MSG; RL-50; Accent; Ajinomoto; Glutacyl; Glutavene; Vetsin

Instruction for use Use a fresh allergen vial each time a new stimulation is performed
Dissolve the allergen prior to use as follows:
Add 250 µl of Stimulation Buffer to vial, vortex briefly
Use allergen solution according to the protocol

Recommendation Additional to the normal concentration of the allergen given, we recommend to stimulate the cells with 1:5 diluted allergen solution
(Mix 40 µl of allergen solution with 160 µl of stimulation buffer)

n Stimulation Up to 4 stimulations could be run with 1 vial

Storage Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial			
2.5 mg			
Conc. in vial			
10 mg/ml			
Conc. in stimulation	2 mg/ml	5 mg/ml	2.27 mg/ml
Cut-off	≥ 70 pg/ml	≥5% CD63 SI* (≥2)	≥5% CD63 SI* (≥2)

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays
*SI (Stimulation Index) is defined as the gross stimulation divided by the negative control value



BÜHLMANN Laboratories AG
Baselstrasse 55
CH-4124 Schönenbuch/Basel
Switzerland

Phone +41 61 487 12 12
Fax orders +41 61 487 12 99
info@buhlmannlabs.ch
www.buhlmannlabs.ch



ADS014-02E_CE621

CAST® is a registered trademark of BÜHLMANN.