



# Cereals

## Food



Name	Order Code	Common Name				
		EN	FR	DE	IT	ES
Wheat	<b>BAG-F4</b>	Blé	Weizen	Grano	Trigo	
Rye Flour	<b>BAG-F5</b>	Seigle	Roggen	Segale	Centeno	
Barley Flour	<b>BAG-F6</b>	Orge	Gerste	Orzo	Cebada	
Oat Flour	<b>BAG-F7</b>	Avoine	Hafer	Avena	Avena	
Maize	<b>BAG-F8</b>	Maïs	Mais	Mais	Maíz	
Rice	<b>BAG-F9</b>	Riz	Reis	Riso	Arroz	
Baker's Yeast	<b>BAG-F45</b>	Levure de bière	Bäckerhefe	Lievito	Levadura reposteria	

**Instruction for use** Use a fresh allergen vial each time a new stimulation is performed  
Dissolve the allergen prior to use as follows:  
Add 250 µl of Stimulation Buffer to vial, vortex briefly  
Use allergen solution according to the protocol

**n Stimulation** Up to 4 stimulations could be run with 1 allergen vial

**Storage** Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial 25 ng			
Conc. in vial 100 ng/ml			
Conc. in stimulation	20 ng/ml	50 ng/ml	22.5 ng/ml
<b>Cut-off</b>	<b>≥ 200 pg/ml</b>	<b>≥15% CD 63</b>	<b>≥15% CD 63</b>

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays