



Fish & Meat



Name	Order Code	Common Name			
		EN	FR	DE	IT
Codfish	BAG-F3	Morue	Dorsch	Merluzzo	Bacalao
Crab	BAG-F23	Crabe	Krabbe	Granchio	Cangrejo
Shrimp	BAG-F24	Crevette	Garnele	Gambero	Gamba
Squid	BAG-F258	Calmar	Tintenfisch	Polipo	Pulpo
Oyster	BAG-F290	Huître	Auster	Ostrica	Ostra
Anisakis	BAG-P4	Anisakis	Anisakis	Anisakis	Anisakis
Pork	BAG-F26	Porc	Schweinefleisch	Maiale	Carne de cerdo
Beef	BAG-F27	Boeuf	Rindfleisch	Manzo	Carne de ternera

Instruction for use Use a fresh allergen vial each time a new stimulation is performed
 Dissolve the allergen prior to use as follows:
 Add 250 µl of Stimulation Buffer to vial, vortex briefly
 Use allergen solution according to the protocol

n Stimulation Up to 4 stimulations could be run with 1 allergen vial

Storage Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial 25 ng			
Conc. in vial 100 ng/ml			
Conc. in stimulation	20 ng/ml	50 ng/ml	22.5 ng/ml
Cut-off	≥ 200 pg/ml	≥15% CD 63	≥15% CD 63

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays