



Seeds/ Beans/ Nuts



A Commitment to Diagnostics

Name	Order Code	Common Name			
		EN	FR	DE	IT
Sesame Seed	BAG-F10	Sésame	Sesam	Seme di sesamo	Sésamo
Peanut	BAG-F13	Arachide	Erdnuss	Arachide	Cacahuete
Soybean	BAG-F14	Soja	Sojabohne	Seme di soia	Soja
Hazelnut	BAG-F17	Noisette	Haselnuss	Nocciola	Avellana
Almond	BAG-F20	Amandes	Mandeln	Mandorla	Almendras
Cashew Nut	BAG-F202	Noix de cajou	Acajounüsse	Noce di acagiù	Nuez de acajú

Instruction for use Use a fresh allergen vial each time a new stimulation is performed
 Dissolve the allergen prior to use as follows:
 Add 250 µl of Stimulation Buffer to vial, vortex briefly
 Use allergen solution according to the protocol

n Stimulation Up to 4 stimulations could be run with 1 allergen vial

Storage Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial 25 ng			
Conc. in vial 100 ng/ml			
Conc. in stimulation	20 ng/ml	50 ng/ml	22.5 ng/ml
Cut-off	≥ 200 pg/ml	≥15% CD 63	≥15% CD 63

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays



BÜHLMANN Laboratories AG
 Baselstrasse 55
 CH-4124 Schönenbuch/Basel
 Switzerland

Phone +41 61 487 12 12
 Fax orders +41 61 487 12 99
 info@buhlmannlabs.ch
 www.buhlmannlabs.ch



ADS014-03E_SBN
 CAST® is a registered trademark of BÜHLMANN