



Weeds

Inhalant Allergen



A Commitment to Diagnostics

Name	Order Code	Common Name			
		EN	FR	DE	IT/ES
Common Ragweed	BAG-W1	Ambrosia elatior	Ambrosie élevée	Beifußblättrige Ambrosie	
Mugwort	BAG-W6	Artemisia vulgaris	Armoise	Beifuss	Assenzione selvatico
Ribwort/ Plantain	BAG-W9	Plantago lanceolata	Plantain lancéolé	Spitzwegerich	Lanciuola
Wall Pellitory	BAG-W19	Parietaria officinalis	Pariétaire	Ästiges Glaskraut	Parietaria officinalis

Instruction for use Use a fresh allergen vial each time a new stimulation is performed
 Dissolve the allergen prior to use as follows:
 Add 250 µl of Stimulation Buffer to vial, vortex briefly
 Use allergen solution according to the protocol

n Stimulation Up to 4 stimulations could be run with 1 allergen vial

Storage Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial			
25 ng			
Conc. in vial			
100 ng/ml			
Conc. in stimulation	20 ng/ml	50 ng/ml	22.5 ng/ml
Cut-off	≥ 200 pg/ml	≥15% CD 63	≥15% CD 63

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays



BÜHLMANN Laboratories AG
 Baselstrasse 55
 CH-4124 Schönenbuch/Basel
 Switzerland

Phone +41 61 487 12 12
 Fax orders +41 61 487 12 99
 info@buhlmannlabs.ch
 www.buhlmannlabs.ch



ADS014-02E_Wee

CAST® is a registered trademark of BÜHLMANN