




Molds

Inhalant Allergen

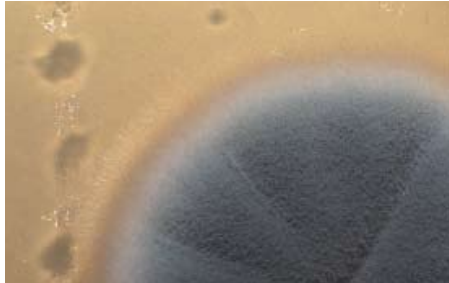

Name	Order Code
Penicillium notatum	BAG-M1
Cladosporium herbarum	BAG-M2
Aspergillus fumigatus	BAG-M3
Candida albicans	BAG-M5
Alternaria tenuis	BAG-M6

Instruction for use	Use a fresh allergen vial each time a new stimulation is performed Dissolve the allergen prior to use as follows: Add 250 µl of Stimulation Buffer to vial, vortex briefly. Use allergen solution according to the protocol.
n Stimulation	Up to 4 stimulations could be run with 1 allergen vial
Storage	Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial 25 ng			
Conc. in vial 100 ng/ml			
Conc. in stimulation	20 ng/ml	50 ng/ml	22.5 ng/ml
Cut-off	≥ 200 pg/ml	≥15% CD 63	≥15% CD 63

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays

