



Paper Wasp

Insects Venoms



A Commitment to Diagnostics

Name	Order Code
	BAG2-I4 <i>Polistes apachus</i>

Instruction for use	Use a fresh allergen vial each time a new stimulation is performed Dissolve the allergen prior to use as follows: Add 250 µl of Stimulation Buffer to vial, vortex briefly Use allergen solution according to the protocol
n Stimulation	Up to 4 stimulations could be run with 1 allergen vial
Storage	Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial			
6.25 ng			
Conc. in vial			
25 ng/ml			
Conc. in stimulation	5 ng/ml	12.5 ng/ml	5.5 ng/ml
Cut-off	≥ 200 pg/ml	≥10% CD 63 ⁺	≥10% CD 63 ⁺

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays



BÜHLMANN Laboratories AG
Baselstrasse 55
CH-4124 Schönenbuch/Basel
Switzerland

Phone +41 61 487 12 12
Fax orders +41 61 487 12 99
info@buhlmannlabs.ch
www.buhlmannlabs.ch



ADS014-02E_I4

CAST® is a registered trademark of BÜHLMANN