



# Storage Mites

## Mixes

Order Code	BAG-DX1	Content per vial/allergen
Components	D70 Acarus siro	6.25 ng
	D71 Lopidoglyphus destructor	6.25 ng
	D72 Tyrophagus putrescentia	6.25 ng
	D73 Glycyphagus domesticus	6.25 ng

Instruction for use	Use a fresh allergen vial each time a new stimulation is performed Dissolve the allergen prior to use as follows: Add 250 µl of Stimulation Buffer to vial, vortex briefly Use allergen solution according to the protocol
n Stimulation	Up to 4 stimulations could be run with 1 allergen vial
Storage	Store unopened allergens at 2-8°C until exp. date

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Content per vial			
25 ng			
Conc. in vial			
100 ng/ml			
Conc. in stimulation	20 ng/ml	50 ng/ml	22.5 ng/ml
<b>Cut-off</b>	<b>≥ 200 pg/ml</b>	<b>≥15% CD 63</b>	<b>≥15% CD 63</b>

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays

