




Ioxitalamate

Contrast Media

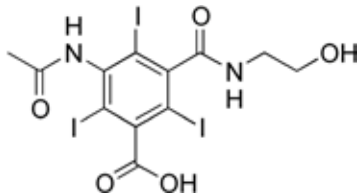

C₁₂H₁₁I₃N₂O₅
MW 643.9

Order Code BAG2-CIOXI

Brands and Generic Names Telebrix®

Instruction for use Use a fresh allergen vial each time a new stimulation is performed;
Dissolve the allergen prior to use as follows:
Add 250 µl of Stimulation Buffer to vial, vortex briefly; use allergen solution according to the protocol

n Stimulations Up to 4 stimulations could be run with 1 vial

Storage Store unopened allergens at 2-8°C until exp. date
Do not store reconstituted allergens

Content per vial 1.25 mg

Conc. in vial 5 mg/ml after reconstitution

	CAST®-2000	Flow-CAST®	Flow2 CAST®
Conc. in stimulation	1 mg/ml	2.5 mg/ml	1.1 mg/ml
Cut-off	n.d.	≥5% CD63; *SI ≥2	≥5% CD63; *SI ≥2

A Commitment to Diagnostics

CAST®-Allergens are intended for in vitro leucocyte stimulation in the CAST®-Assays
*SI (Stimulation Index) is defined as the gross stimulation divided by the negative control value



BÜHLMANN Laboratories AG
Baselstrasse 55
CH-4124 Schönenbuch/Basel
Switzerland

Phone +41 61 487 12 12
Fax orders +41 61 487 12 99
info@buhlmannlabs.ch
www.buhlmannlabs.ch



ADSO14-04E_CIOXI

CAST® is a registered trademark of BÜHLMANN.