Extraction Kit

for BÜHLMANN fCAL[®] tests

B-CAL-EX3

B-CAL-EX12

Revision date: 2017-09-27

BÜHLMANN LABORATORIES AG

Baselstrasse 55
CH - 4124 Schönenbuch, Switzerland
Tel.: +41 61 487 1212
Fax: +41 61 487 1234
info@buhlmannlabs.ch

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ENGLISH

INTENDED USE

The extraction buffer (B-CAL-EX) is a reagent designed for the extraction of calprotectin in human stool specimens using the BÜHLMANN Smart-Prep or manual weighing method for laboratory analysis with BÜHLMANN fCAL® tests.

REAGENTS SUPPLIED

Code	Reagents	Quantity	Preparation
B-CAL-EX3	Extraction Kit extraction buffer	3 bottles x 125 mL	Ready to use
B-CAL-EX12	Extraction Kit extraction buffer	12 bottles x 125 mL	Ready to use

Table 1

MATERIALS REQUIRED BUT NOT PROVIDED

- Smart-Prep (Code: B-CAL-RD)
- 10 µL disposable inoculation loops
- 15 mL polypropylene tubes with screw caps required for standard extraction procedure
- Multi tube vortex mixer
- Precision balance (10-150 mg)
- Micro centrifuge (≥3000 g)

REAGENT STORAGE AND STABILITY

Do not freeze reagents! Shelf life of unopened/ opened reagents at 2-8 °C: see expiry date on package label.

WARNINGS AND PRECAUTIONS

- Read carefully the instructions prior to the use of the buffer. Buffer performance will be adversely affected, if the reagent is incorrectly stored as detailed in this instruction for use.
- Components must not be used after the expiry date printed on the labels.
- Let the reagent adjust to room temperature and mix well (vortex) before use.

Extraction

- To receive quantitative results it is important to homogenize the stool sample completely with the extraction buffer.

SPECIMEN COLLECTION AND HANDLING

If the extraction devices are used, less than 1 g of native stool specimen is needed for the extraction procedure.

Collect stool samples into plain tubes.

Important: The sample must be collected without any chemical or biological additions in the collection device.

Specimen Transport

Stool specimens should be received by the laboratory within 3 days of collection. The specimens may be transported at room temperature (23 °C).

Specimen Storage

Received stool specimens should be stored at 2-8 °C and extracted within 3 days.

Extract Storage

Calprotectin in extracts obtained by manual weighing method is stable at 2-8 °C for 6 days or at -20 °C for 18 months.

STOOL SAMPLE EXTRACTION

With the BÜHLMANN Smart-Prep

Refer to the corresponding Smart-Prep (Code: B-CAL-RD) Instruction for Use (IFU).

With the manual weighing method ¹⁾

1. Label and weigh (tare) the empty polypropylene tube together with the inoculation loop.
2. Take out 50 to 100 mg of the stool sample by means of the inoculation loop and place it into the pre-weighted tube.
3. Calculate the net amount of sample, break off the inoculation loop and leave the lower part of the loop in the tube.
4. Add Extraction Buffer according to the formula
$$\frac{x \text{ mg stool} \times 49}{\mu\text{L extraction buffer}} = y$$
(e.g. 50 mg stool x 49 = 2450 µL buffer)
to the tube and close the tube
5. Homogenize the sample on a multi tube vortexer by vigorous shaking (at highest speed) for 30 minutes.
6. Transfer the homogenate into a 2 mL Eppendorf tube and centrifuge in a microcentrifuge for 5 minutes at 3000 x g.
7. Take the supernatant into a fresh, labelled tube and continue with the assay procedure of the corresponding BÜHLMANN kit.

Important: The extraction buffer (B-CAL-EX) should only be used in combination with BÜHLMANN fecal calprotectin tests.

SHIPPING DAMAGE

Please notify your distributor, if this product was received damaged.

REACH

None of the materials and reagents in the kit require a Safety Data Sheet (MSDS) according to CLP-Regulation (EC) No 1272/2008 and directive EC 1907/2006 (REACH).

¹⁾ According to the IFU of the BÜHLMANN fCAL® ELISA and Quantum Blue® fCAL tests.

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