CALEX® Cap Collection Set

B-CALEX-CCSET25, Release date:2020-09-02 Version A2


For *in vitro* diagnostic use

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


A CALEX® Cap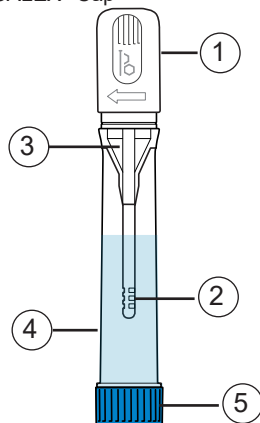

B

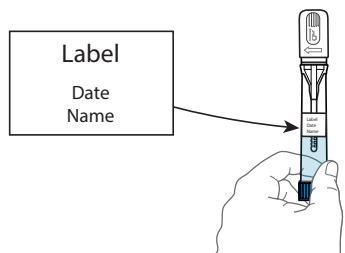C

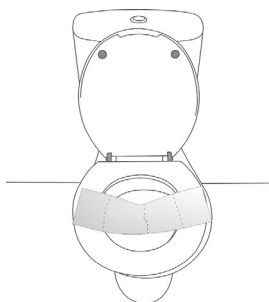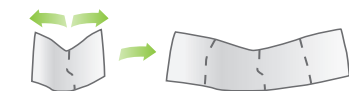ENGLISH

INTENDED USE

The CALEX® Cap Collection Set allows patients to directly collect a defined amount of human stool sample for subsequent analysis with BÜHLMANN fecal marker assays.

MATERIALS AND ACCESSORIES

Quantity	Components
1	CALEX® Cap device filled with extraction buffer (5 mL)
2	Stool collection sheets
1	Label
1	Instruction for use
1	Resealable bag

Store refrigerated at 2-8 °C.

SAFETY PRECAUTIONS AND IMPORTANT ADVICE

- **Never open the blue cap (A,5).**
- The device must not be used after the expiry date printed on the labels.
- The device can be shipped at ambient temperature.
- The device must not be reused.
- If you have problems to collect the stool sample i.e. the stool does not stick to the grooves of the CALEX® Cap device, perform the stool collection on another day. Use a new CALEX® Cap Collection Set.
- Children should be assisted by a parent.

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PROCEDURE

1. Please empty your bladder first if necessary, since urine can affect the test.
2. Label the CALEX® Cap device with date and name (B).
3. Unfold the stool collection sheet by holding the open ends and gently pulling outwards (C). Place the stool collection sheet on top of the toilet seat near the back (C).

CAUTION: The sheet must not touch the toilet water.

4. Make sure that the stool collection sheet is correctly fixed on the toilet seat.

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



ITALIANO

PORTUGUÊS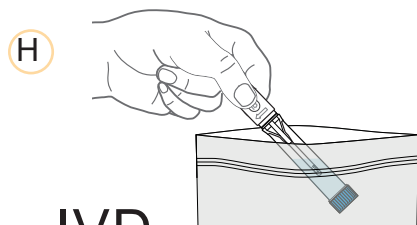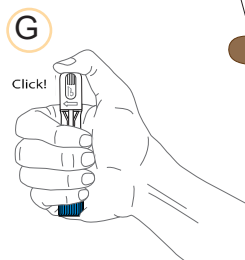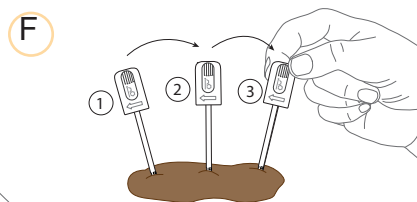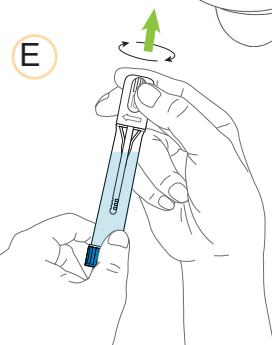

- 5 Make sure that your stool sample is caught on the stool collection sheet (D).

NOTE: In case your stool collection fails the first time, you have a second stool collection sheet provided with the collection set.

- 6 Hold the CALEX® Cap device with the white cap at the top and remove the white sampling pin (A,1) by simultaneously turning clockwise and pulling it upwards (E).

- 7 Dip the dosing tip (A,2) with the grooves into the stool, and twist before removing. Repeat the procedure at 3 to 5 different positions of the stool sample in order to fill the grooves of the dosing tip completely (F).

CAUTION: Make sure that all grooves are completely filled with stool. It does not matter if the dosing tip of the sampling pin is fully covered with stool, since excess stool will be stripped off in the next step.

- 8 Place the sampling pin (A,1) back into the funnel (A,3) of the CALEX® Cap body (A,4) and push it into the final locking position. You feel and hear a "click" (G).

NOTE: After the stool sample has been collected, you can flush the remaining stool down the toilet together with the stool collection sheet.

- 9 Shake the CALEX® Cap device vigorously for 5 seconds.

- 10 Put the CALEX® Cap device into the resealable bag (H). Keep the CALEX® Cap device in a cool, shaded place until you can return it or send it to the address you were given.

NOTE: If CALEX® Cap shipment is necessary, make sure it is safely stored in the resealable bag and ask your provider for suitable UN3373 packaging.

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