

Quantum Blue® Anti-Adalimumab

anti-drug antibody
rapid test

Anti-ADM antibody
results within
minutes



- Lateral flow rapid test for anti-adalimumab antibody determination
- Time to result 15 minutes
- High correlation to established third party ELISA assays
- Comprehensive monitoring together with the trough level test



Quantum Blue® Anti-Adalimumab anti-drug antibody rapid test

Key features

- Fastest assay for qualitative anti-adalimumab antibody measurements on the market
- Immediate results for near-patient settings
- Combination of drug levels and anti-drug antibodies on the most popular rapid test platform Quantum Blue®. The complete TDM solution
- Drug-sensitivity allows for measurement of free antibodies
- High agreement of 91.7% to established third party ELISA assays

		RIDASCREEN Category (≥2.5ng/mL)		
		negative	positive	total
Quantum Blue® Anti-Adalimumab (0.2 µg _{eq} /mL)	negative	46	14	60
	positive	6	174	180
	total	52	188	240

Fig.1

Agreement = 91.7%

Measurement of 240 patient samples with the Quantum Blue® Anti-Adalimumab rapid test and an established reference ELISA assay reveals an excellent **agreement of 91.7%** (Fig.1).



BÜHLMANN Laboratories AG
Baselstrasse 55
4124 Schönenbuch
Switzerland

Phone +41 61 487 12 12
Fax orders +41 61 487 12 99
info@buhlmannlabs.ch
www.buhlmannlabs.ch

Quantum Blue® is a registered trademark
of BÜHLMANN in many countries.

Ordering codes:

LF-ADAD10 10 cassettes
LF-ADAD25 25 cassettes

Related products:

LF-ADIF10 / 25 10 / 25 cassettes
LF-TLIF10 / 25 10 / 25 cassettes
LF-TLAD10 / 25 10 / 25 cassettes
LF-CALE25 25 cassettes
LF-CHR25 25 cassettes

☑️-marked products