

Quantum Blue® Anti-Infliximab

anti-drug antibody
rapid test

Anti-IFX antibody
results within
minutes



- Lateral flow rapid test for anti-infliximab antibody determination
- Time to result 15 minutes
- Excellent specificity of 94.9% for anti-IFX antibody detection
- High agreement to established third party ELISA assays



Quantum Blue® Anti-Infliximab anti-drug antibody rapid test

Key features


- Fastest assay for qualitative anti-infliximab antibody measurements on the market
- Excellent specificity and sensitivity for anti-IFX antibody detection
- Combination of drug levels and anti-drug antibodies on the most popular rapid test platform Quantum Blue®. The complete TDM solution
- Drug-sensitivity allows for measurement of free antibodies
- High agreement of 88% to established third party ELISA assays

Measurement of 238 patient samples with the Quantum Blue® Anti-Infliximab rapid test and an established reference ELISA assay reveals an excellent **agreement of 88%** (Fig.1).

	RIDASCREEN Category 119 ng/mL			
	negative	positive	total	
Quantum Blue® Anti-Infliximab 1.3 µg _{eq} /mL	negative	112	23	135
	positive	6	97	103
	total	118	120	238

Fig.1

Agreement = 88%

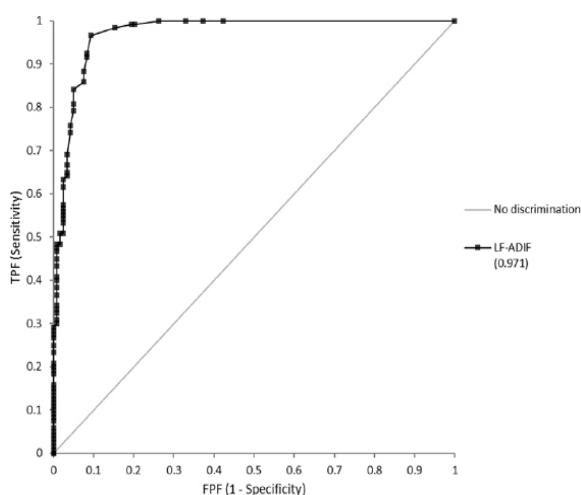

ROC curve analysis reveals **excellent specificity of 94.9%** and **sensitivity of 80.8%** at cut-off 1.3 µg_{eq}/mL with Area Under the Curve (AUC) = 0.971 (Fig.2).

Fig.2



BÜHLMANN Laboratories AG
Baselstrasse 55
4124 Schönenbuch
Switzerland

Phone +41 61 487 12 12
Fax orders +41 61 487 12 99
info@buhlmannlabs.ch
www.buhlmannlabs.ch

Quantum Blue® is a registered trademark of BÜHLMANN in many countries.

Ordering codes:

LF-ADIF10 10 cassettes
LF-ADIF25 25 cassettes

Related products:

LF-ADAD10 / 25 10 / 25 cassettes
LF-TLIF10 / 25 10 / 25 cassettes
LF-TLAD10 / 25 10 / 25 cassettes
LF-CALE25 25 cassettes
LF-CHR25 25 cassettes

CE-marked products